

SOMACorrect™

*Peyronie's
Correction Therapy*

SOMACorrect™

CONTENTS

One piece, ultra-ease pumping mechanism on pump head with simple twist-lock attachment to outer cylinder (C). Concentric cylinder inserts, (A) and (B), and cushioned inserts. Travel case, instruction manual, 142g tube of lubricant. One year warranty. 100% Refundable within 90 days minus £40 handling charge.

£209.00 + £8.00 P&P



UK and RoI Distributor

iMEDicare Ltd
Unit 11 Shakespeare
Industrial Estate
Shakespeare Street
Watford WD24 5RR
T +44(0)1923 23 7795
F +44(0)1923 804206
www.iMEDicare.co.uk
contact@imedicare.co.uk

SomaTherapy ED® SOMACorrect™			
	Distributors: iMEDicare Ltd Unit 11 Shakespeare Industrial Estate Shakespeare Street Watford WD24 5RR Tel: +44(0)1923 23 7795 www.iMEDicare.co.uk		
	Manufacturer: Asputa Medical Systems LLC 1227 Broad St, Augusta, GA 30906, United States Tel: 800(705) 825-3600 www.asputa.com		
	SOMACorrect (15000)		

Size to Fit (STF)

iMEDicare



SOMACorrect™

Peyronie's Correction Therapy

SOMACorrect Peyronie's Correction Therapy is the revolutionary NEW treatment regimen for men with the penile curvature (and possible erectile dysfunction) associated with Peyronie's disease.

This condition, which affects up to 3% of men, involves the spontaneous formation of scar tissue (sometimes painful) or a hardened plaque on the penile erectile tissue.



When severe enough this can lead to significant penile deformity or curvature. While the plaque itself is benign (harmless) - it can cause a great deal of distress if the level of penile curvature is sufficient to interfere with normal sexual intercourse.

By placing the penis in one of the appropriate size adapted circumferential cylinders and using vacuum therapy pressure to forcibly induce a series of 'straight' erections over a 15-20 minute period each day for up to 26 weeks - this increased blood flow and 'stretch' of the scar tissue plaque can eventually lead to a reduction in penile curvature and an improvement in erectile health and function in most cases.

Note that currently, SOMACorrect is not available on NHS Prescription.



Size to Fit (STF)


	Width (cm)	Height (cm)
A	203.2	57
B	203.2	44
C	203.2	56
D	203.2	63
A/B	274.9	83

Mechanisms of Action?

Regular stretching and straightening may burst an element of the collagen fibres that make up the scar tissue (plaque area) and help to (A) 'RESHAPE' the plaque thereby reducing its contractile impact (degree of Mechanical Distortion) and the degree of penile curvature. It is theorised that regular SOMACorrect may cause (B) PARTIAL FRAGMENTATION of the plaque itself (particularly when diffuse in nature) with some element of plaque (C) SURFACE REABSORPTION and an (D) INCREASE IN VASCULARIZATION of plaque tissue. (E) EARLY TREATMENT OF ASSOCIATED ERECTILE DYSFUNCTION (ED) in relevant patients may be of benefit. Possibly, mechanical tissue expansion principles of action with subsequent stimulated (F) CELL

(TISSUE) PROLIFERATION may also apply with regular SOMACorrect application increasing tissue mass on the interior curved side of the penis:

FORCE + TIME = GROWTH / tissue expansion: sustained mechanical stress inducing increased biosynthetic activity of the shorter side as it is subject greater traction. A high degree of calcification may reduce the pliability and scope for revascularization of plaque tissue - therefore vacuum therapy application is particularly indicated during the active phase of Peyronie's disease and certainly pre-calcification if possible, although response rates may be influenced by plaque location / shape / size / patient's potential for stimulated cell proliferation and compliance in procedural use, and possible discomfort.

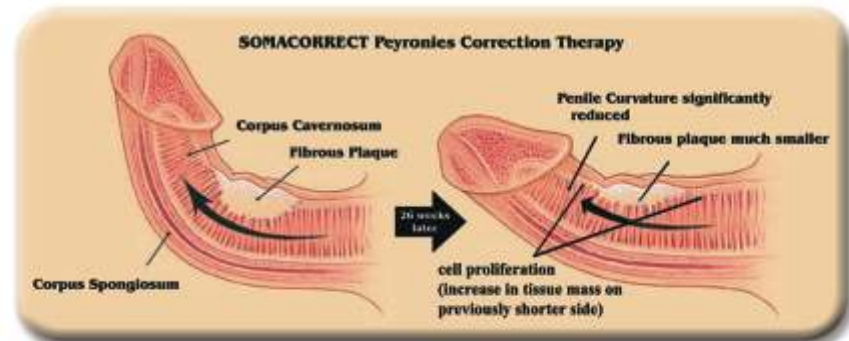

In some cases, a vacuum induced erection captured with an erection maintenance ring is 'artificially' less curved in light of the extra-corporal filling effect so long as the ring is held in place.

